

Flexible Partial Solutions

Valplast®

- Esthetics: Pink shades that allow your patient's natural tissue tone to appear through the material.
- Comfort: Thin, lightweight and flexible.
- Retention: Flexes into a retentive position below the undercut.
- Strength: Clinically unbreakable, more durable than acrylic.
- Ease: No tooth or tissue preparation is required.

Indications:

- Flexible and easy insertion
- Design versatility
- Acrylic allergies
- History of partial frame breakage
- Alternative to implants or fixed restoration
- Presence of tori

Contraindications:

- Need for more rigid appliance due to poor soft tissue support or few remaining teeth
- Quality of teeth is too poor to support Valplast
- Minimal vertical clearance, less than 3mm
- Immediate denture applications
- Flat ridge

EconoFlex (DuraFlex®)

- Easy: Simple to adjust and polish.
- Esthetics: Lifelike pink shades with veined acrylic detail.
- Comfortable: Thin, lightweight, and flexible.
- Strength: Clinically unbreakable and more durable than acrylic.
- Hygienic: Won't absorb stains or odors.
- Easy patient care.
- FDA Approved acetyl resin material.

Indications:

- Flexible and easier to insert
- Design versatility
- Acrylic allergies
- History of partial frame breakage
- Alternative to implants or fixed products
- Presence of tori

Contraindications:

- Need for more rigid appliance due to poor soft tissue support or few remaining teeth
- Quality of teeth is too poor to support DuraFlex
- Minimal vertical clearance, less than 3mm
- Immediate denture applications
- Flat ridge

Lucitone® FRS®

- Esthetics: Invisible clasps, four Lucitone 199® shades and clear
- Comfort: True flexibility, monomer and metal-free
- Durable: High tensile strength for better retention, high flexural strength for more stability

Indications:

- Partial
- Unilateral Partial
- Combination Metal/ FRS Partial



ClearFrame®

- Metal-free esthetics & increased patient comfort.
- Vertical stability with occlusal rests.
- Reline, repair & polish acrylic saddles chairside.
- Hygienic, will not absorb stains or odors.
- No metal taste.

Indications:

- Most cast partial indications
- Long and free end saddles
- Flat ridge
- Few remaining teeth

Contraindications:

- Class II or Class II mobility of abutment teeth
- Short clinical length or lack of adequate undercut
- Acrylic allergies
- Minimal vertical clearance less than 4mm



Call for a case pick-up today!

(320) 252-2070 ♦ (800) 899-1115 ♦ 540 Progress Rd., Waite Park, MN 56387 ♦ www.thoeledental.com